

**COVIDIEN***positive results for life™*

Kangaroo Joey™ Enteral Feeding Pump

Set Loading Instructions for Use

Follow proper procedures for filling or spiking feed container. Press power button in lower right-hand corner to turn pump on.



Grasp thumb tab on valve and insert firmly into left pocket. Press down on thumb tab to ensure valve is fully seated. Thumb tab should align with slot in blue door. Blue door will not close if tab is not fully seated.



Grasp black ring retainer, gently wrap tubing around rotor, insert retainer directly into right pocket. **DO NOT OVERSTRETCH TUBING.**



Once the feeding set is loaded, close the door. The pump is now ready for normal operation.

**COVIDIEN**

COVIDIEN, COVIDIEN with logo and
™ marked brands are trademarks of Covidien AG or its affiliate.
©2008 Covidien AG or its affiliate. All rights reserved.

H6255 2.5M 0507

15 HAMPSHIRE STREET
MANSFIELD, MA
02048

1-800-962-9888
1-508-261-8000

WWW.COVIDIEN.COM



COVIDIEN

positive results for life™

Kangaroo Joey™ Enteral Feeding Pump

Hang Height Instructions for Use



In order to achieve proper accuracy the initial fluid line in the feeding set container must be 6 inches above the top of the feeding pump when initiating the feeding cycle.



COVIDIEN

COVIDIEN, COVIDIEN with logo and
™ marked brands are trademarks of Covidien AG or its affiliate.
©2008 Covidien AG or its affiliate. All rights reserved.

H-6255-5M-0307

15 HAMPSHIRE STREET
MANSFIELD, MA
02048

1-800-962-9888
1-508-261-8000

WWW.COVIDIEN.COM